

Instructions for Use/Assembly Instructions

SC6015A/SHB-U

Shower Bench Adjustable U-Shape Seat

Intended use:

Height-adjustable shower chair with back, ideal for persons who require seating in a shower or individuals struggling with mobility issues

To assemble:

- Attach the 2 brackets to the seat as shown below, by attaching corresponding screws, and washers found on the silver tubing
- Push the 4 leg extensions/stubs up into the shafts and adjust the height evenly all around - ensure the legs are all adjusted on the same height as indicated by the no's on the stubs
- Secure backrest to frame underneath where the brackets cross over as shown below using the shortest screws
- Weight Capacity – 100kg's

