

## ***Instructions for Use/Assembly Instructions***

### **SC6020D/B-STOOL**

## **Bath Stool Round with Swivel Seat**

### **Intended use:**

This Height-adjustable round bath stool with swivel seat is ideally suited for its purpose, providing support and stability for those who struggle with stability. This product can be used in the shower or bath

### **To assemble:**

- Attach the 2 brackets to the seat as shown below, by attaching corresponding screws, and washers found on the silver tubing
- Push the 4 leg extensions/stubs up into the shafts and adjust the height evenly all around - ensure the legs are all adjusted on the same height as indicated by the no's on the stubs
- Weight Capacity – 100kg's



Attach 2 brackets to the swivel seat and secure with screws and washers as shown

Attached the 4 Leg extension stubs to the 2 brackets and adjust height evenly all round

