

MAXIMUM WEIGHT CAPACITY - 100KG

FS965LH/B-ROL4

1. Fold open Rollator by pushing down on folding support bracket.
2. Place removable basket under seat.
3. Secure foam backrest in holes
4. Pull loop brake lever upwards to check if brakes are working.
5. Push loop brake levers downwards till click is heard (wheels should be locked).
6. Ensure that brakes are always locked when sitting on the seat.
7. To fold for storage, remove the basket, fold up footrest and pull rod joining folding support bracket upwards.

