

# ***Instructions for Use/Assembly Instructions***

## ***FS560/OT***

### Overbed Table - Dual Top

#### **Intended use:**

Our over-bed table provides a solid surface, that is the height of convenience. Literally! Easily maneuverable, on 4 roll-easy castor wheels. Simply roll it into your position of choice, adjust the height to suit, and tilt the table to your preference.

#### Features:

- Ideal for use in the bedroom or the lounge
- Height and tilt adjustment, to suit user requirements
- The smaller table section to the side remains in a fixed position, providing a stable independent surface
- Raised ridge to ensure items remain on the table surface when tilted
- Requires 8cm space under the bed

#### To Assemble

1. Undo the Black Round Knob from the upright pole and pull out the silver tube
2. Using the longer screws, attach the upright with the triangular metal base to the bottom base of the table
3. Slide the silver tube down the upright securing it with the round black knob
4. Use the 4 shorter screws to connect the silver base plate to the smaller fixed side tray
5. Use the lever on the side of the smaller fixed side tray to swivel the tabletop and to secure it into position
6. All screws and allen keys are included

