

## ***Instructions for Use/Assembly Instructions***

### **SC7010W/COMMODE**

#### **Commode Drop Arm with Wheels**

##### **Intended use:**

This **commode with drop arms and wheels** is a practical and portable hygiene solution designed to assist individuals with limited mobility. Whether used as a **bedside toilet**, **over-toilet chair**, or **transfer commode**, it offers dependable support and improved independence.

##### **Assembly Instructions**

- Push the 4 leg extensions/stubs up into the shafts and adjust the height evenly all around - ensure the legs are all adjusted on the same height
- Secure backrest to frame by sliding it into the two shafts beneath the commode and adjust backrest by clipping the bronze pin into the desired position
- Insert Grey Commode bucket with lid
- Clip on Grey Lid and Seat
- Weight Capacity – 100kg's

