

Instructions for Use/Assembly Instructions

FS908LJ Wheelchair Series

Instructions for Use

- Push down on rail guides of Seat to open chair (Ensure your fingers are not under the seat when opening chair to prevent injury)
- Fit detachable footrest by lining it up on the front brackets of the chair and the black clip on chair - clip into position
- Adjust footrest height with spanner supplied in backrest pocket
- To fold up chair, pull up from underneath the seat
- Flip back armrest by releasing black clip on side of armrest
- Unity brakes can be used by handler if user is unable to operate the brakes
- User may propell chair by using pushrims
- Max Weight Capacity – 100 kg's

