

Instructions for Use/Assembly Instructions

FS952LCQ/WCL-ELE

Lightweight Wheelchair with Elevating Footrests

Intended use:

Lightweight wheelchair with comfort, flexibility, and daily usability.

Features elevating footrests, flip-up adjustable armrests, quick-release wheels, and padded seat. Foldable and transport-ready with anti-tip safety.

Elevating leg rests on a wheelchair allows for adjusting the angle and height of the footplates to support the legs and improve comfort. To use them, first, ensure the wheelchair is stable by engaging the brakes. Then, locate the release lever on the leg rest and push or pull it to unlock the mechanism. Adjust the leg rest to the desired height and angle and then release the lever to lock it in place. To swing the leg, rest out of the way for transfers, locate the release lever on the inside of the leg rest and swing it outwards.

Features

- Elevating and Detachable Footrest
- Flip Up and Adjustable Armrest
- Folds Up
- 24" Quick Release Rear Wheels
- Anti Tippers for Amputee Use
- Padded Insert

Specifications

- Aluminium Frame
- Seat Width – 18" (45cm)
- Weight Capacity – 100kg's
- Weight of Chair – 17.5kg's
- 24" Pneumatic Rear Wheels
- 6" Solid Castors
- Floor to Armrest – 72cm
- Floor to Push Handles – 91cm
- Floor to Seat – 50cm

