

Instructions for Use/Assembly Instructions

SC6055AN/ S-CH NB

Shower Chair No Back Adjustable Shower Chair (White, Silver)

Intended use:

Height-adjustable shower chair with no back, ideal for persons recovering from hip operations and individuals struggling with mobility issues

To assemble:

- Attach the brackets to the seat as shown below, by attaching corresponding screws, and washers found on the silver tubing
- Push the leg extensions/stubs up into the shafts and adjust the height evenly all around - ensure the legs are all adjusted on the same height as indicated by the no's on the stubs
- Attached Left & Right Armrest by clipping it into the grey housing on the side of the chair – ensure the clip is secure before using the Shower Chair
- Weight Capacity – 100kg's

