

Instructions for Use/Assembly Instructions

SC7060H/RTS-SF

2-in-1 Raised Toilet Seat and Safety Frame

Intended use:

A must-have aid for anyone who has trouble getting up from the toilet. This simple addition rests securely atop a standard toilet, raising the existing height of your toilet seat by 17cm. Arms provide added stability and security whilst standing and lowering. Armrests are removable or flip up and out of the way for ease of transfer.

To assemble

- Remove current toilet seat from toilet bowl
- Attach Raised Toilet Seat to toilet bowl by securing the seat with screws underneath the toilet bowl -
- Ensure the unit is secured safely and sturdy before use -
- Armrests can be flipped back if needed
- Weight Capacity – 100kg's

