

Instructions for Use/Assembly Instructions

SC7060K/PU & SC7060K/Moulded seat

Raised Toilet Seat (PU Soft Seat) & (Molded Hard Seat)

Intended use:

This Height-adjustable raised toilet seat with soft seat, is ideal for post-hip surgery or those struggling with limited mobility. Raise seat height from 11cm to 18.5cm. Soft padding for extra comfort. Fits standard toilets. Also available with the Standard Hard Blow Molded seat

To assemble

- Remove or lift current toilet seat
- Attach Raised Toilet Seat to toilet bowl by securing the seat onto the toilet bowl rim – (fits only the SA standard oval toilet bowl
- Ensure the unit is secured safely and sturdy before use -
- Weight Capacity – 100kg's

