

FS7962L.SH.FOLD/ALUMINIUM FOLDABLE SHOWER CHAIR U-SHAPE WITH WHEELS

1.Foldable frame with release turning knob for easy storage when not in use and transporting:

- **Step 1**
To unfold the frame, join the top and lower frame together into the lower curved shaped bracket. Turn the knob completely in by turning the knob clockwise making sure it is tightened correctly.
- **Step 2**
To fold the frame, turn the knob anti clockwise until it is completely removed. Lift the top frame up until its released from the curved bracket and then drop down in folded position.

2. To fit the backrest

- **Step 1**
Press the side pins in on both sides and attach into the frame by pushing it down till it clicks in.
- **Step 2**
To remove the backrest push both pins in and pull out.

3.Lockable 4-inch front castor wheels for safety when used in shower and 6-inch rear castors:

- To activate the front brakes, press the lever down with your foot.
- To release the brakes, lift the lever up with your foot.

3.Flip up armrests for easy transfer.

4.Flip up footrests for easy storage and transporting.

5.Maximum Weight capacity 100kg.

