

FS804LABJP/LIGHTWEIGHT OXIDIZED ALUMINIUM FOLDABLE TRANSIT WHEELCHAIR EXTRA NARROW 14.5"

1. Oxidized aluminium foldable frame:

- **To unfold the wheelchair push down on the side bars of the seat until it clips down on to the frame, secondly lift the back rest up until you hear a click sound that indicates that it's attached to the frame.**

2. Maintenance-free solid front and rear tyres. 5" rubber solid front castor wheels and 8" rubber solid rear wheels. Shock-absorbing wheel design for a smoother ride

3. Caregiver-Friendly Controls - equipped with rear push handles and hand-operated brakes, allowing caregivers to navigate and control the wheelchair with ease and confidence.

Aluminium alloy brakes and handbrake to lock rear wheels:

- **To lock the rear wheels lift the lever of the brakes towards the handle and then press the smaller lever on the brakes down.**
- **To release the handbrake, press the small lever up.**

4. **Flip up armrests** for easy transfer.

5. **Flip up footrest** for easy transfer.

6. **Foldable frame** making it easier for traveling:

- **Step 1: To fold the wheelchair pull the centre of the footrest up and then pull the centre of the seat towards you.**
- **Step 2: Drop down the back by pressing the back bottom lever behind the back rest towards the frame in order to fold the back.**

7. **The leg strap goes behind your calves to prevent your feet from sliding off the footrest.**

8. Durable Oxford nylon upholstery, reinforced with high-tensile inner canvas sleeves

9. **Maximum Weight Capacity 100kg**